Tracheo Tape soft plus

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PZN XXXXXXX

JJ-MM

UDI
(xx)xxxxxxxxxxxx
(xx)xxxxxx
(xx)xxxxxx

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GB Intended use | indication | description

The SERVOX® Tracheo Tape soft plus is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open.

The SERVOX® Tracheo Tape soft plus secures the aid in the tracheostoma and prevents it from accidentally slipping out

- Reverse side of tube holder with cotton coating
- Suitable for all age groups depending on the neck circumference
- single-use, do not reuse (risk of infection)
- The maximum duration of use is 48 hours, after dispose of in household waste
- Can be seamlessly adjusted to the circumference of the neck
- Soft padding to prevent decubitus

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GB Application

- 1 Place the band around the back of the neck
- 2 Insert both hooks into the eyelets on the neck flange from below. The back of each hook is facing the throat.
- 3 Use the slider to adjust the length of the band.
- 4 Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- 5 To remove, unhook one side of the band from the neck flange.

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GB Please note

- Regularly check the fit and adjust where necessary.
- Remeasure the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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