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SERVOX[®]Tracheo Nose Neo O₂

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INSTRUCTION MANUAL

SERVOX[®]Tracheo Nose Neo O₂

O₂ Adapter for the SERVVOX[®] Tracheo Nose Neo F/P
(Artificial Noses) for Neonates

CE 0197

MD

Contents

1 Foreword	10
1.1 Intended purpose	10
1.2 Indications	11
1.3 Contraindications	11
1.4 Side effects	11
2 Safe use	11
3 Description	12
4 Application	13
5 Cleaning, storage and transport	13
6 Order information	14
7 Symbols used	14
8 Disclaimer	14

1 Foreword

Dear customer,

We would like to thank you for your trust and the purchase of our quality product.

This instruction manual is intended for patients, relatives over 16 years of age and trained medical staff. The SERVOX® Tracheo Nose Neo O₂ can be used at home and in a clinical setting. Prior instruction on use by trained medical staff is required. Application should be in line with the descriptions in this instruction manual and in accordance with the recognised rules of medicine.

Note

Read this instruction manual carefully. It contains important information on the safe and proper use of the product. Keep this instruction manual for future reference.

A batch number is assigned to the SERVOX® Tracheo Nose Neo O₂ for unique traceability. The batch number is indicated on the packaging next to "LOT". If you have any questions or problems with the SERVOX® Tracheo Nose Neo O₂, please contact your field representative or Servona GmbH, quoting the product's batch number

1.1 Intended purpose

The SERVOX® Tracheo Nose Neo O₂ can be attached to the SERVOX® Tracheo Nose Neo P or SERVOX® Tracheo Nose Neo F artificial noses and enables extra oxygen to be administered via the oxygen connection.

Note

The SERVVOX® Tracheo Nose Neo O₂ is a single patient product and must not be reused by other people. There is a risk of infection.

1.2 Indication

The SERVVOX® Tracheo Nose Neo O₂ is placed on the SERVVOX® Tracheo Nose Neo artificial nose to provide oxygen.

The SERVVOX® Tracheo Nose Neo O₂ must not be used for ventilation.

1.3 Contraindication

- Allergies to the material polycarbonate (PC)

1.4 Side effects

There are no known side effects provided when the patient uses the product properly and observes the safety instructions.

2 Safe use

Safe and proper handling is important for the patient's health! Ignoring the safety instructions can have significant effects on your health or even cause life-threatening situations.

Warning: The word "Warning" indicates a situation that, if not avoided, can result in grave injury or even death.

Please observe the following safety instructions:

Warning

Risk of injury: If the SERVVOX® Tracheo Nose Neo O₂ shows signs of damage (such as sharp edges, tears or similar) or if the shape is altered, injuries may result.

- Do not use a damaged SERVVOX® Tracheo Nose Neo O₂!
- The SERVVOX® Tracheo Nose Neo O₂ must be replaced immediately, even if damaged only slightly!
- Do not make any changes to the SERVVOX® Tracheo Nose Neo O₂ yourself!

Risk of suffocation/risk of respiratory distress: Accidentally or permanently blocking the tracheostoma can result in breathlessness or suffocation of the patient.

- Never cover or block the front opening of the SERVVOX® Tracheo Nose Neo O₂ with clothing or similar!
- Never use a wet SERVVOX® Tracheo Nose Neo O₂. Moisture can penetrate the filter of the SERVVOX® Tracheo Nose Neo or a moist adapter can slip off the SERVVOX® Tracheo Nose Neo.

Warning

Risk of ignition/ risk of fire and explosion if extra O₂ is administered: O₂ increases the flammability of materials and helps a fire to originate and spread.

- Be sure to read all safety instructions in connection with the devices you use for extra O₂ administration!
- Do not use the SERVOX® Tracheo Nose Neo O₂ near sources of ignition such as electrical devices or naked flames such as candles.
- Make sure that oxygen cannot accumulate under clothing or bed textiles!

Risk of allergy: Remove the SERVOX® Tracheo Nose Neo O₂ immediately at the first signs of an allergic reaction, such as a skin rash, redness, breathlessness or rapid heart!

Note

Please inform us or the responsible authorities if you experience a life-threatening illness or injury or permanent physical damage as a result of using our product, or require hospital treatment or have to extend hospital treatment due to using our product.

3 Description

The SERVOX® Tracheo Nose Neo O₂ is an adapter for extra oxygen administration (O₂ administration) and is attached to the front of the SERVOX® Tracheo Nose Neo. A 4 mm oxygen connection is integrated on the front protruding edge of the ring-shaped adapter. An oxygen source, such as an oxygen concentrator or an oxygen cylinder, can be connected there via an O₂ safety hose. The oxygen is thus routed directly in front of the filter of the SERVOX® Tracheo Nose Neo, enriching the inhaled air with oxygen.

Note

Extra oxygen administration must be prescribed by the treating doctor.

The SERVOX® Tracheo Nose Neo O₂ is supplied non-sterile and is made from PC free of latex and DEHP without metallic or electrically conductive materials. There are no restrictions on use during magnetic resonance tomography (MRT). The SERVOX® Tracheo Nose Neo O₂ can be used for a single patient for a maximum period of 7 days if the instructions for use are followed.

The SERVOX® Tracheo Nose Neo O₂ can be disposed of with household waste.

The packaging can be recycled. From the date of manufacture, the SERVOX® Tracheo Nose Neo O₂ has a shelf life of 5 years if the package is unopened and stored correctly.

4 Application

Wash and dry hands thoroughly before use.

Check prior to use: Check the packaging and adapter for damage. Do not use if the packaging or adapter are damaged.

Attaching the **SERVOX® Tracheo Nose Neo O₂**: Push the adapter firmly onto the front of the SERVOX® Tracheo Nose Neo so that it cannot fall off. Any SERVOX® Tracheo Nose Neo O₂ that has fallen off must be disposed of or cleaned before reuse.

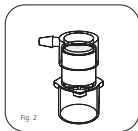
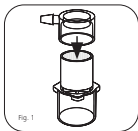
O₂ connection: Push an O₂ safety hose for 4 mm connections firmly onto the 4 mm oxygen connection with a slight twisting motion so that it cannot come loose unintentionally. Then connect the other end of the hose to the oxygen source (O₂ concentrator, O₂ bottle, etc.).

The tube always needs sufficient clearance so that it does not exert any pull on the SERVOX® Tracheo Nose Neo. The filter opening must never be covered by clothing or other textiles. The O₂ safety hose can be pulled off the 4 mm oxygen connection again with a slight twisting movement.

Note

Always observe the generally applicable safety instructions for the use of oxygen.

Remove the SERVOX® Tracheo Nose Neo O₂ adapter: Pull off the SERVOX® Tracheo Nose Neo with a slight twisting motion and clean immediately.

5 Cleaning, storage and transport

After each use, wipe the SERVOX® Tracheo Nose Neo O₂ thoroughly inside and outside with a lint-free cloth moistened with warm, clean water and allow to dry completely.

Then store or transport in a clean, dry environment free of dust and protected from light at temperatures between +5 and +25°C. Protect from direct sunlight and heat.

6 Order information

Designation	PU	Article no.
SERVOX® Tracheo Nose Neo O ₂ Adapter	4	29660
SERVOX® Tracheo Nose Neo P	30	30027
SERVOX® Tracheo Nose Neo F	30	28391

7 Symbols used

	CE mark with the identification number of the notified		Order number
	Follow the instructions for use		Batch number
	Keep away from sunlight		Central pharmaceutical number
	Do not use if the packaging is damaged		Date of manufacture
	Temperature limit		Manufacturer
	Keep dry		Dual collection and recycling system
	Non-sterile		medical device
	single-patient product		

8 Disclaimer

In the event of damage, the manufacturer does not assume liability in case of non-compliance with the instruction manual or the safety instructions or manipulation of the product.

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