SERVOX® Kombi-Tracheokomprese AL standard, gelocht

Orbisana
GESUNDHEITSWELT

90 x 100 mm

- EN** SERVOX® Combi Tracheal Compress AL, standard, with punched hole
- FR** SERVOX® Comresse trachéale combinée AL, standard, perforée
- IT** SERVOX® Compressa tracheale combinata AL, standard, perforata
- ES** SERVOX® Compresas traqueales combinadas AL, estándar, perforadas
- BG** SERVOX® Комбиниран трахеален компрес AL, стандартен, с отвори
- HU** SERVOX® standard kombinált légsző-seprárna AL, lyukas
- EL** Συνδυάζομενη επίθεμα τραχειοστομίας SERVOX® Kombi AL, στάνταρ, με στή
- HR** Kombinirana trahealna kompres SERVOX®, AL, standardna, perforirana
- NL** SERVOX® Combi-tracheokompres AL standaard, geperforeerd
- PT** Compressa traqueal combinada SERVOX® AL standard, perfurada
- PL** Kombinowany kompres tracheostomijny SERVOX® AL standard, z otworami
- SL** Kombinirana standardna trahealna kompresza z luknjо SERVOX® AL

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EN

Intended purpose SERVIX® Kombi Tracheal Compress AL, standard, with punched hole provide cushioning for the neck flange and absorb moisture that secretes from the tracheostoma to prevent skin irritation and infections.

Indication Tracheostomised or laryngectomised tracheal cannula users with heavy secretions.

Description The wipeable, impermeable layer protects clothing from becoming soaked in case of heavy secretions. The punched hole ensures a secure fit and prevents slipping. The aluminium coating helps prevent the compress from sticking to the skin. The punched hole ensures a more secure fit and prevents slipping.

Application Slide the tracheal cannula through the punched hole of the compress (with the aluminium coating to the tracheostomy) until the neck flange is well cushioned. Then insert the tracheal cannula as usual. To remove, pull out the tracheal cannula as usual, pull the compress off the cannula tube and dispose of it.

– **Changing frequency:** with every cannula more frequently as needed, at least every 24 hours. – If the skin is irritated, use a compress with no aluminium coating and consult a doctor.

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