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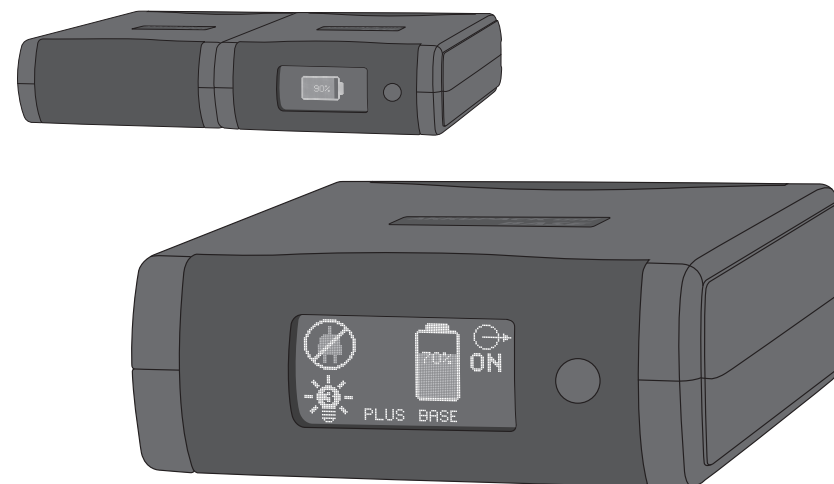
AKKUPACK uni BASE / PLUS

User's manual

©2023 HOFFRICHTER GmbH
All rights reserved.

The content published in this user's manual is the sole property of the HOFFRICHTER GmbH. All forms of reproduction, editing, distribution and any kind of exploitation, even in part, require the prior written consent of the HOFFRICHTER GmbH.

The HOFFRICHTER GmbH reserves the right to amend or replace this user's manual without prior notice.

Please ensure that you are always working with the most current version of this user's manual. Should you have any questions, please contact the device provider, or check our information at www.hoffrichter.de.

SERIAL NUMBER

Every HOFFRICHTER GmbH device is supplied with a serial number for traceability purposes.

Please enter your device's serial number here. You will find the serial number on the rating plate on the bottom of the device.

SN

BASE / PLUS: /

Please always quote the serial number for all queries and complaints.

KONFORMITÄT

With the CE mark, the manufacturer documents compliance with all rules and thus the minimum legal standards.


CONTENTS

Scope of delivery	43
AKKUPACK uni BASE “Ventilation”	43
AKKUPACK uni BASE “Respiratory therapy”	44
General	45
Information on user’s manual	45
Symbols on the rating plate	45
Symbols on the bag	45
Symbols used in this user’s manual	46
Safety information	47
Intended use	48
Description of device	49
Commissioning	50
AKKUPACK uni BASE	50
AKKUPACK uni PLUS	52
Charging the AKKUPACK uni	53
Connection of the AKKUPACK uni to the mains power	53
Normal charging	54
Quick charging	55
Display accuracy of the charge status	55
Connection of devices	56
CARAT ventilator	56
CARAT pro ventilator	57
TRENDvent ventilator	57
Respiratory therapy devices	58
Using the AKKUPACK uni	59
Display	59
Start screen	59
Standby screen	59
Standard screen	60
Error screen	60
Meaning of the symbols on the Standby screen	61
Meaning of the symbols on the Standard screen	62
Operating the device	63
Setting the display brightness	64
Displaying the device identification and charge status	64
Battery life	65
Separation of BASE and PLUS	66
Transport and mobile use	67

Troubleshooting	68
Cleaning and disinfection	70
Maintenance	71
Disposal	71
Device	71
Packaging.....	71
Article numbers	72
Scope of delivery	72
Optional accessories.....	72
Technical data	73
Manufacturer's declaration on electromagnetic compatibility	74
Disclaimer	74



SCOPE OF DELIVERY

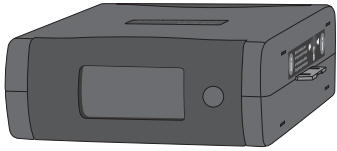AKKUPACK uni BASE “VENTILATION”


BASE base unit

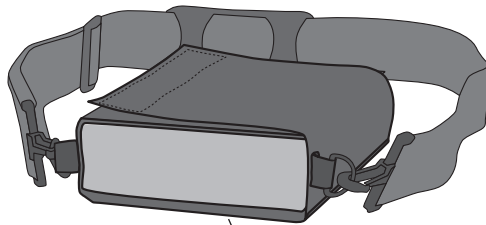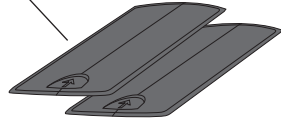Side cover, 2x

BASE ever-ready bag

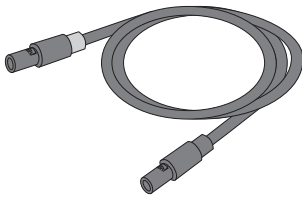Respiratory therapy DC adapter

User's manual

“Power supply unit”
adapter cable

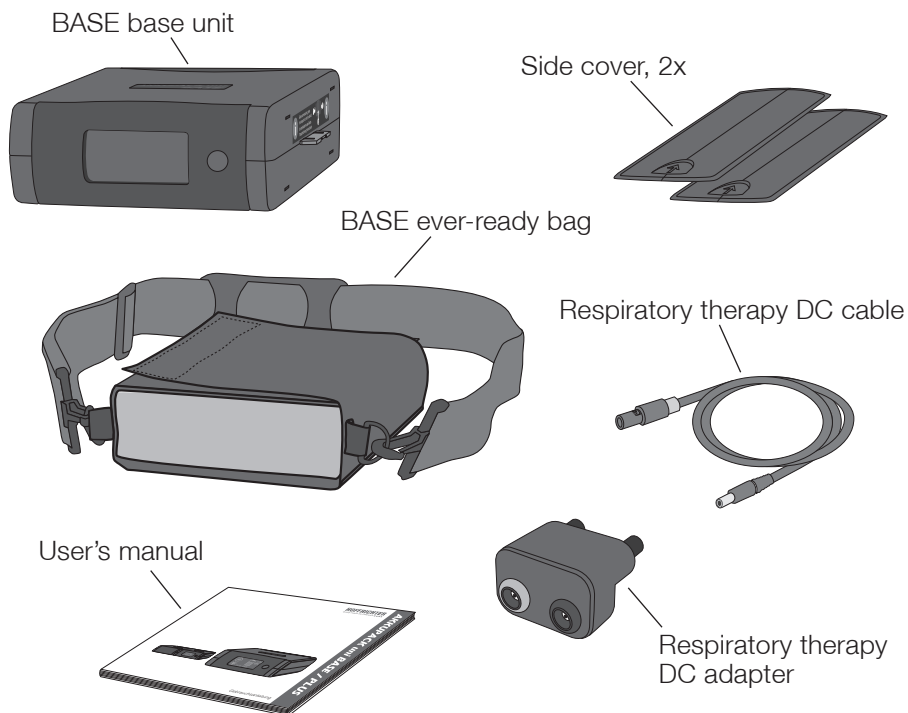
“Device”
adapter cable


“Ventilation” adapter cable set

AKKUPACK uni BASE “RESPIRATORY THERAPY”

AKKUPACK uni PLUS

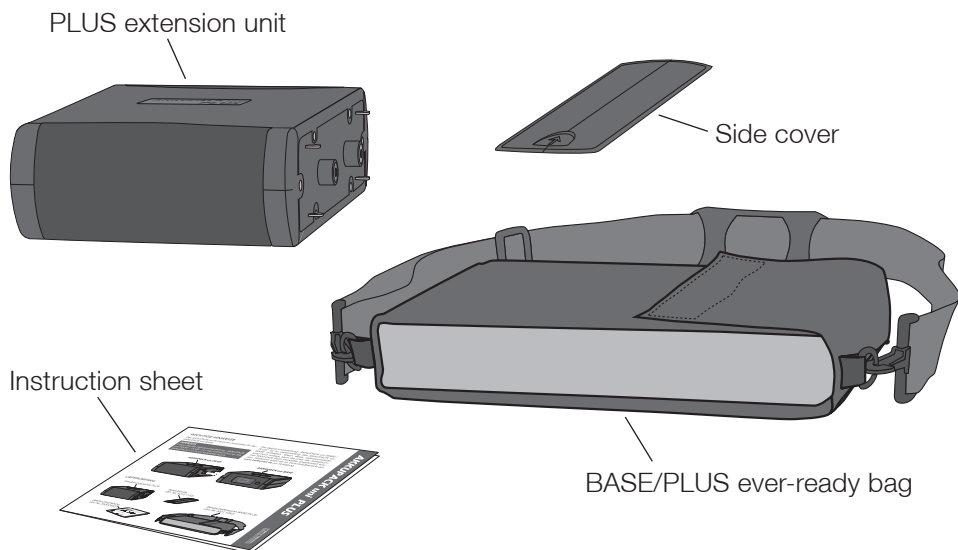GENERAL

INFORMATION ON USER'S MANUAL

Read this user's manual through carefully before using the ventilator for the first time.

Follow the safety and cleaning instructions in particular.

Keep the manual in a safe place close to the device so that you can refer to it immediately if necessary.

SYMBOLS ON THE RATING PLATE

Follow the user's manual.

Manufacturer

Serial number

CE conformity declaration

Protection class II

IP20

The device is protected against solid foreign bodies with a diameter of at least 12.5 mm.

Keep dry!

Do not dispose of the device in the household waste. Please contact the relevant customer services department to find out how to dispose of the device properly.

SYMBOLS ON THE BAG

IP02

The ever-ready/functional bag is protected against dripping water when tilted up to 15°.

SYMBOLS USED IN THIS USER'S MANUAL

Important information is denoted by symbols in this user's manual. Be sure to follow these instructions in order to avoid accidents, personal injury and material damage.

In addition, the local accident prevention regulations and general safety regulations in force in the area of use must be observed.

This symbol denotes general safety instructions. Follow these instructions to avoid accidents, personal injury or material damage.

CAUTION

This symbol denotes hazardous situations that may lead to serious injuries or death.

NOTICE

This symbol denotes information, tips and instructions for the efficient, error-free use of the device.

SAFETY INFORMATION

- The AKKUPACK uni may only be used together with HOFFRICHTER respiratory therapy devices and ventilators (see “Intended use” on page 48).
- The AKKUPACK uni housing does not offer protection against the ingress of water and must never be exposed to rain, moisture or humidity. This could result in malfunction and damage. The AKKUPACK uni must therefore always be in the ever-ready/functional bag, while being transported and operated outdoors.
- Do not place containers with liquid on the device.
- Do not short-circuit the AKKUPACK uni. This could result in damage.
- Only use accessories that have been approved by HOFFRICHTER for use with the AKKUPACK uni.

INTENDED USE

The AKKUPACK uni is an external lithium ion battery system which comprises a base unit (AKKUPACK uni BASE) and an optional extension unit (AKKUPACK uni PLUS).

The AKKUPACK uni can be used as a standby power source for the following device groups:

- TRENDvent
- CARAT from the serial no. C0711300
- CARAT pro
- TREND II
- TREND III
- LAVI
- VECTOR et
- point, point 2
- TREND 110, 200, 210, 300, 400, 500
- AutoTREND
- TRENDlife
- TRENDevent
- all identical OEM products of the here listed devices

The AKKUPACK uni BASE can be used on its own or as a system in combination with the AKKUPACK uni PLUS.

The AKKUPACK uni is suitable for repetitive use.

DESCRIPTION OF DEVICE

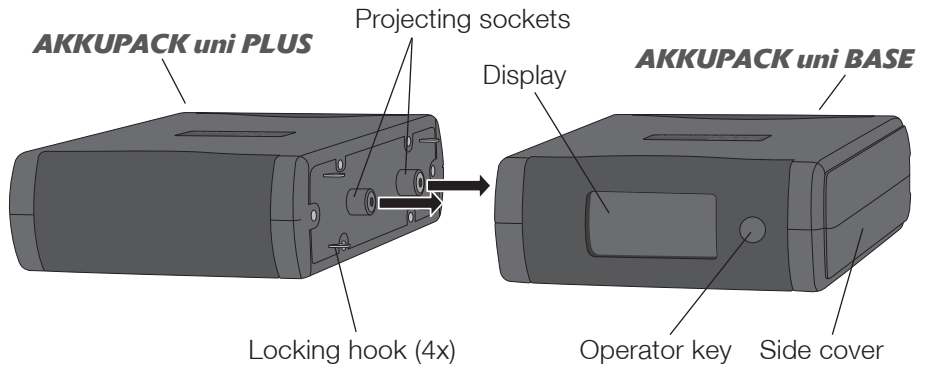
Figure 1: View from front

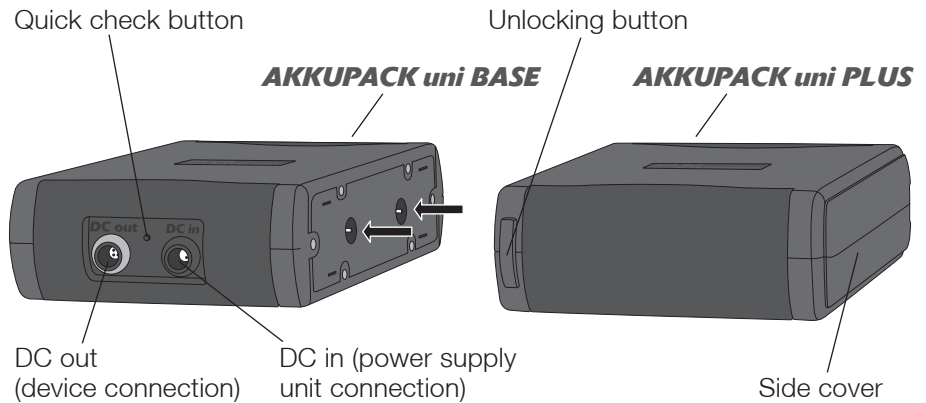
Figure 2: View from rear

COMMISSIONING

⚠ CAUTION

Do not use the device if the housing or any cable or the power supply unit are damaged.

Do not open the device. Repairs may only be carried out by personnel authorized by HOFFRICHTER.

AKKUPACK uni BASE

The AKKUPACK uni BASE includes a transport securing feature. Before using for the first time, this feature must be pushed into the AKKUPACK as shown in the images below and the side cover must be fitted.

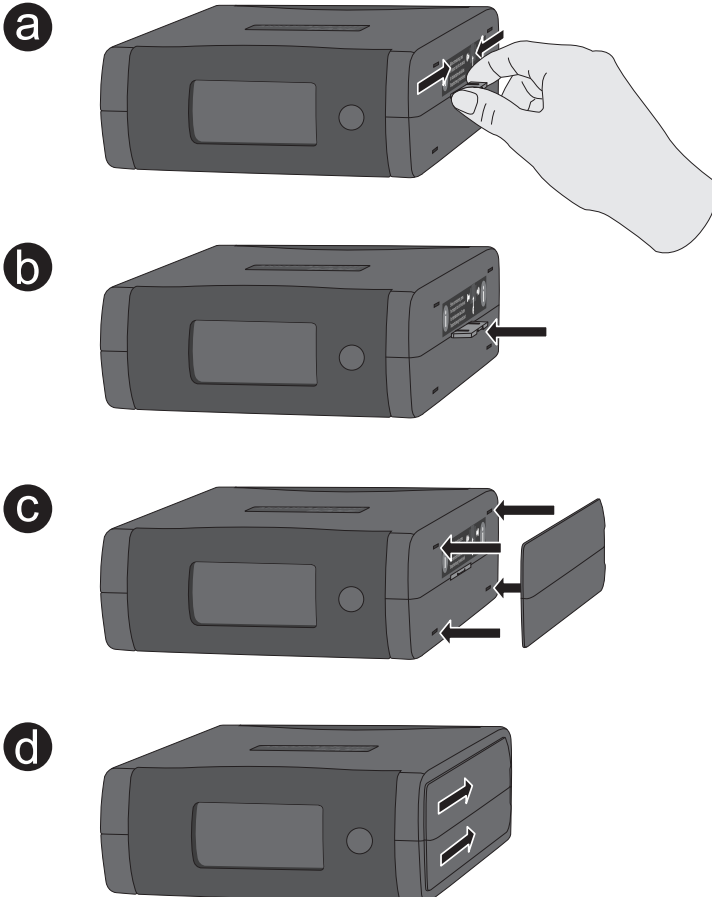
Figure 3: Push in the transport securing feature and fit the side cover

If you wish to use the AKKUPACK uni BASE without the AKKUPACK uni PLUS, fit the second side cover as below:

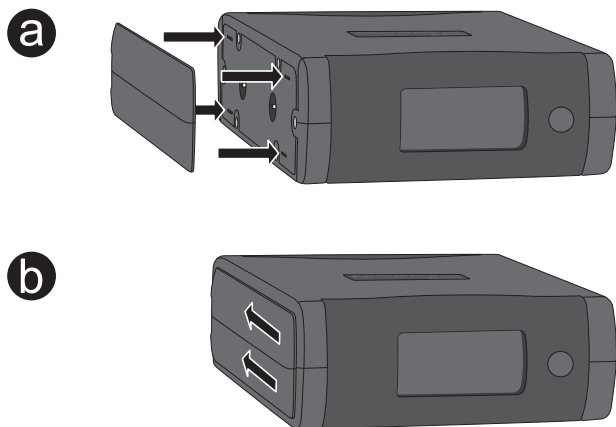
Figure 4: Fit the left-hand side cover

AKKUPACK uni PLUS

Before you can connect the AKKUPACK uni PLUS to the AKKUPACK uni BASE, it may be necessary to remove the left-hand side cover from the AKKUPACK uni BASE.

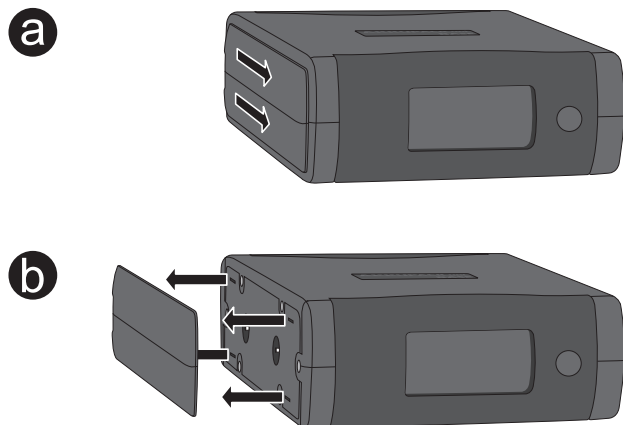
Figure 5: Remove the left-hand side cover from the AKKUPACK uni BASE

The AKKUPACK uni PLUS includes a transport securing feature. Before using for the first time, this feature must be pushed into the AKKUPACK as shown in Figure 3 on page 50 and the side cover included must be fitted on the left-hand side. Then connect the AKKUPACK uni PLUS to the AKKUPACK uni BASE as per Figure 6.

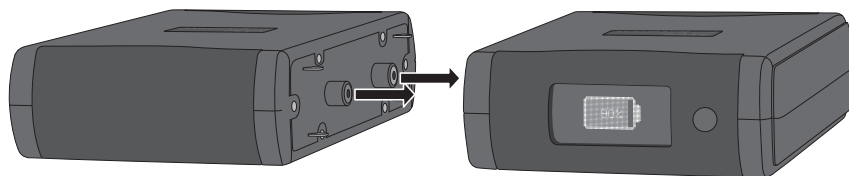
Figure 6: Connect the AKKUPACK uni BASE and the AKKUPACK uni PLUS to each other

CHARGING THE AKKUPACK uni

CONNECTION OF THE AKKUPACK uni TO THE MAINS POWER

The AKKUPACK uni BASE should be fully charged before using for the first time. The charging times are shown in Table 1 on page 54. In order to charge the AKKUPACK uni PLUS, connect it to the AKKUPACK uni BASE as shown in Figure 6 on page 52.

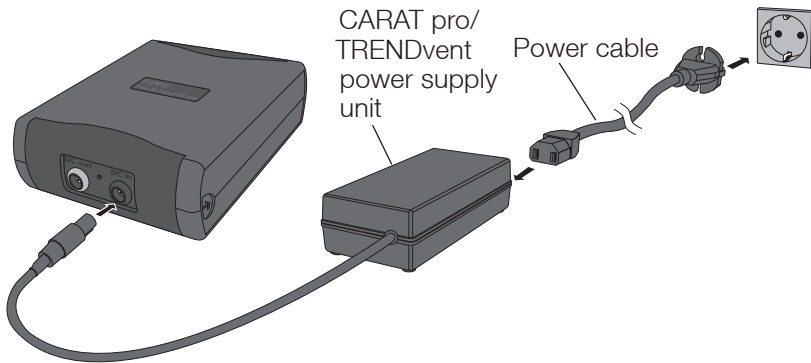
Figure 7: Connection to the CARAT pro/TRENDvent power supply unit (view from rear)

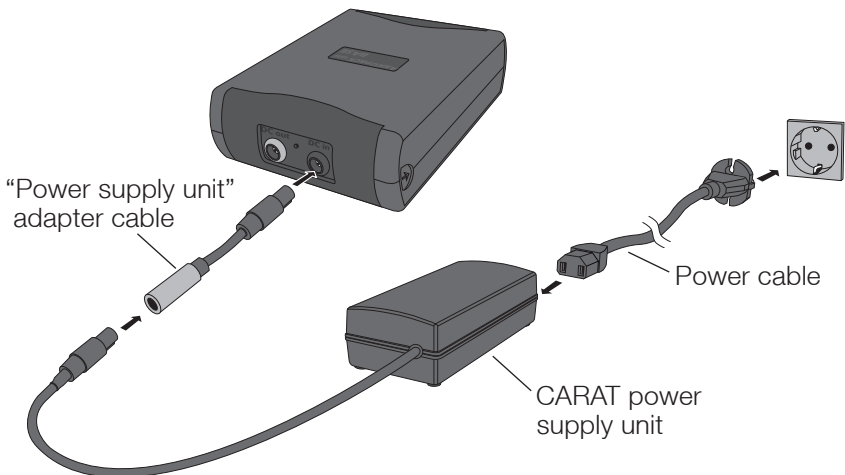
Figure 8: Connection to the CARAT power supply unit (view from rear)

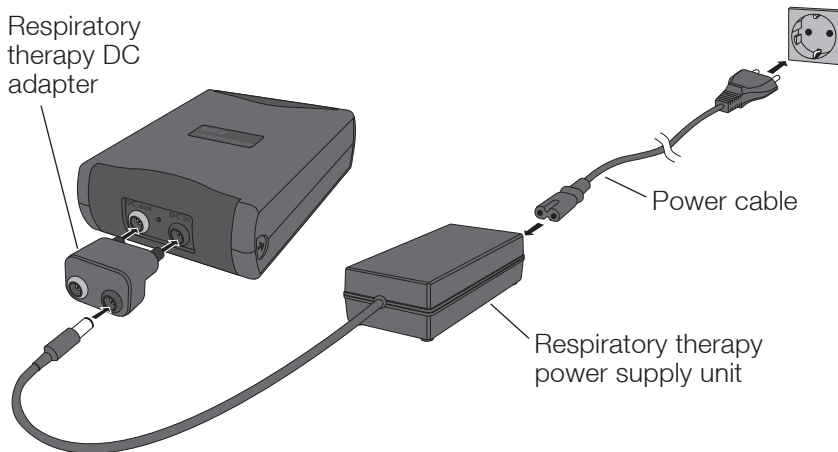
Figure 9: Connection to the respiratory therapy power supply unit (view from rear)

NORMAL CHARGING

CARAT (pro)/TRENDvent power supply unit

If the AKKUPACK uni is supplied with mains voltage and a device is connected at the same time, the AKKUPACK uni is charged normally. In such cases, the AKKUPACK uni BASE is charged first, followed by the AKKUPACK uni PLUS. If, during charging, the connected device is removed from the AKKUPACK uni BASE, there is no switchover to quick charging.

Respiratory therapy power supply unit

HINWEIS

If the AKKUPACK uni is charged via a respiratory therapy power supply unit, there must be no device connected to the AKKUPACK uni BASE.

If the AKKUPACK uni is supplied with mains voltage, the AKKUPACK uni is charged normally. In such cases, the AKKUPACK uni BASE is charged first, followed by the AKKUPACK uni PLUS.

Charging times with a discharged battery	BASE	BASE and PLUS
to 90 %	< 6 h	< 12 h
to 100 %	< 7.5 h	< 15 h

Table 1: Charging time with normal charging (all power supply units)

QUICK CHARGING

NOTICE

If the AKKUPACK uni is charged via a respiratory therapy power supply unit, quick charging is not possible.

If the AKKUPACK uni is supplied with mains voltage and no device is connected, the AKKUPACK uni is charged quickly.

If, during charging, a device is connected to the AKKUPACK uni BASE, a switch is made to normal charging.

Charging times with a discharged battery	BASE	BASE and PLUS
to 90 %	< 3 h	< 3 h
to 100 %	< 4 h	< 4 h

Table 2: Charging times with quick charging (CARAT (pro)/TRENDvent power supply unit only)

DISPLAY ACCURACY OF THE CHARGE STATUS

Charging	Discharging
0 % – 100 % in increments of 10 %	100 % – 5 % in increments of 5 % 5 % – 0 % in increments of 1 %

Table 3: Display accuracy of the charge status

CONNECTION OF DEVICES

As soon as the power supply unit or a device is connected to the AKKUPACK uni BASE it switches on automatically. If the power supply unit and connected device are removed, the AKKUPACK uni BASE turns off automatically.

CARAT VENTILATOR

The AKKUPACK uni is charged during ventilation with the connection option shown below. In case of a power failure the AKKUPACK uni supplies the power to the ventilator.

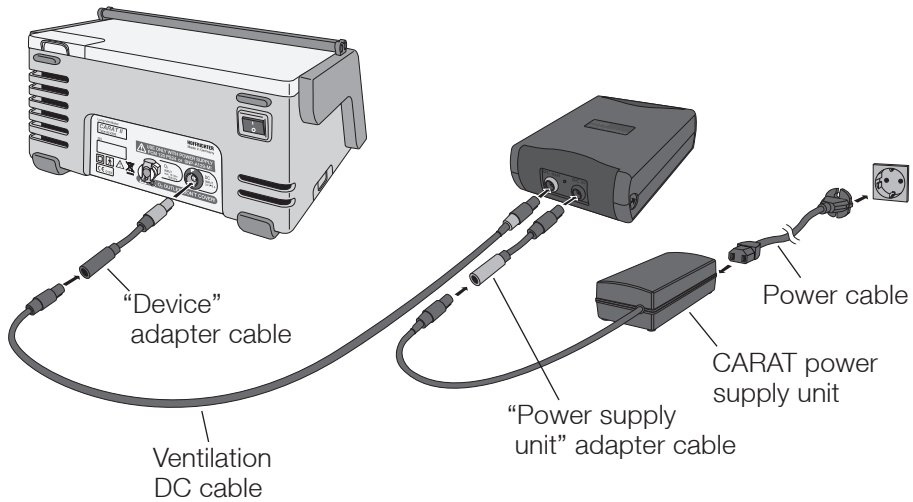
Figure 10: Connection to the CARAT ventilator (view from rear)

CARAT pro VENTILATOR

The AKKUPACK uni is charged during ventilation with the connection option shown below. In case of a power failure the AKKUPACK uni supplies the power to the ventilator.

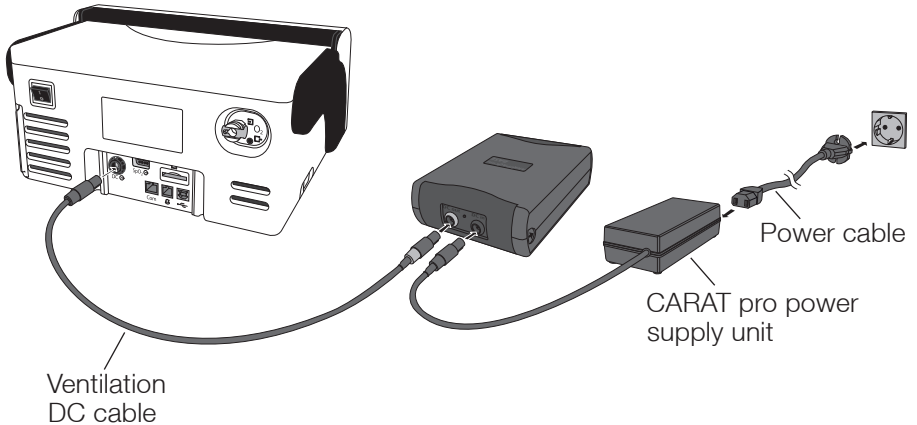
Figure 11: Connection to the CARAT pro ventilator (view from rear)

TRENDvent VENTILATOR

The AKKUPACK uni is charged during ventilation with the connection option shown below. In case of a power failure the AKKUPACK uni supplies the power to the ventilator.

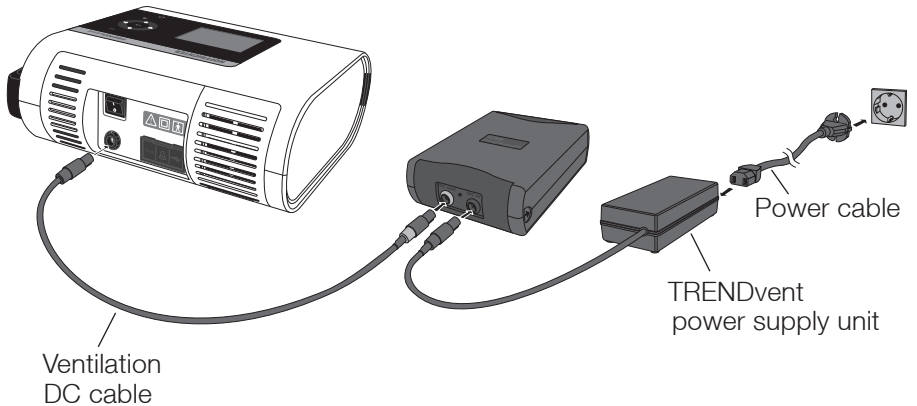
Figure 12: Connection to the TRENDvent ventilator (view from rear)

RESPIRATORY THERAPY DEVICES

The connection options given below apply for all respiratory therapy devices from HOFFRICHTER as well as identical OEM products.

Connection option 1 (battery operation only)

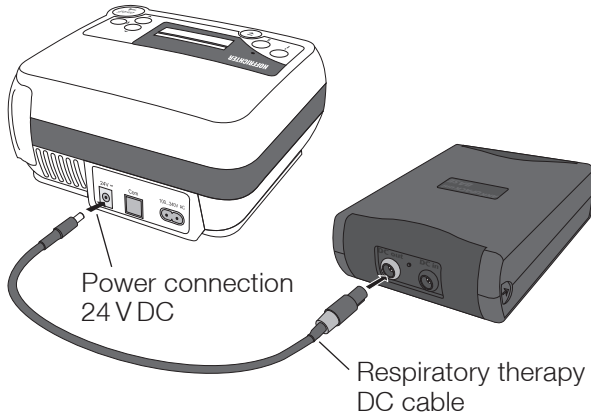
Figure 13: Connection to a respiratory therapy device (view from rear, device given as an example)

Connection option 2 (mains and battery operation)

With connection option 2, the "respiratory therapy DC adapter" must be used in order to ensure proper operation of the respiratory therapy device. The AKKU-PACK uni is not charged with this connection option. In case of a power failure the AKKUPACK uni supplies the power to the respiratory therapy device.

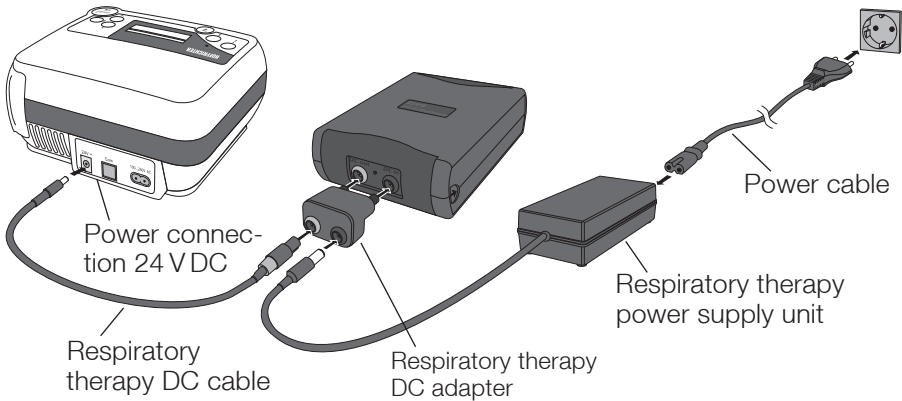
Figure 14: Connection to a respiratory therapy device and the power supply (view from rear, device given as an example)

USING THE AKKUPACK uni

DISPLAY

The AKKUPACK uni BASE display can show four screens:

- Start screen
- Standby screen
- Standard screen
- Error screen

START SCREEN

The Start screen is displayed when the AKKUPACK uni BASE:

- is connected to a device or
- the power supply or
- when the quick check button is pressed.

The serial number and the software version of the AKKUPACK uni BASE are displayed on the Start screen for 3 seconds. When an AKKUPACK uni PLUS is connected, the serial number and the software version of the AKKUPACK uni PLUS are also displayed.

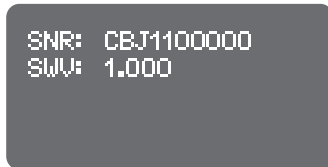
Figure 15: BASE start screen (example)

STANDBY SCREEN

If there are no errors and the Standard screen has not been activated, the Standby screen is shown. It provides information on the charge status and whether the AKKUPACK uni is charged or discharged. Further information on the Standby screen can be found on page 61.

Figure 16: Standby screen (example)

STANDARD SCREEN

The Standard screen is displayed if the operator key is pressed whilst the Standby screen is shown. The Standby screen is shown again after 30 seconds or upon pressing the operator key. The symbols on the Standard screen are described on page 62.

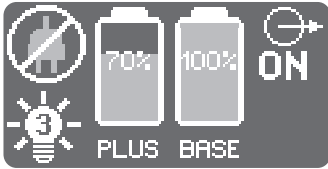
Figure 17: Standard screen (example)

ERROR SCREEN

The Error screen is shown when an error occurs. The Error screen displays a warning symbol and an error code. More information on possible errors and their elimination can be found in the chapter “Troubleshooting” from page 68.

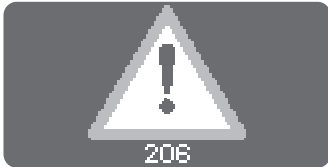
Figure 18: Error screen (example)

MEANING OF THE SYMBOLS ON THE STANDBY SCREEN


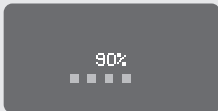
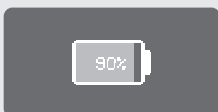
Display of the charge status	Display brightness	Connection to mains voltage / Charging of the batteries?
	Level 1	No
	Level 1	Yes
	Level 2 or 3	No
 <p>  The battery symbol switches between displaying the current charge status and 100 % at regular intervals </p> 	Level 2 or 3	Yes

Table 4: Standby screen

NOTICE

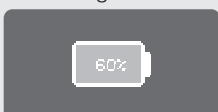When BASE and PLUS are used together, the mean value from these two charge statuses is displayed.

MEANING OF THE SYMBOLS ON THE STANDARD SCREEN

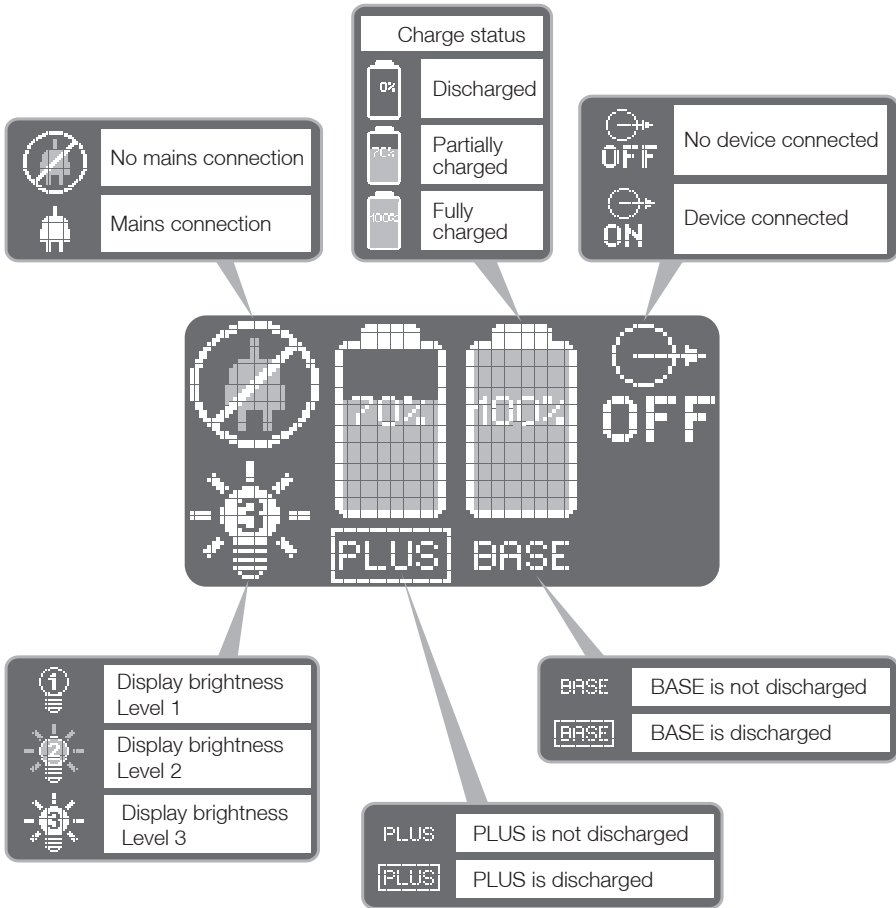
Figure 19: Meaning of the symbols on the Standard screen

OPERATING THE DEVICE

The AKKUPACK uni BASE features 2 control elements:

- The **operator key** on the front of the device:
 - ⇒ To set the display brightness while the Standard screen is shown
 - ⇒ To change between the Standby/Standard and Error screens (only in the event of an error)
 - ⇒ To switch to the next error on the Error screen (only if there are several errors)
- The **Quick check button** on the rear of the unit:
 - ⇒ Display of device identification and the charge status with no device or power supply unit connected

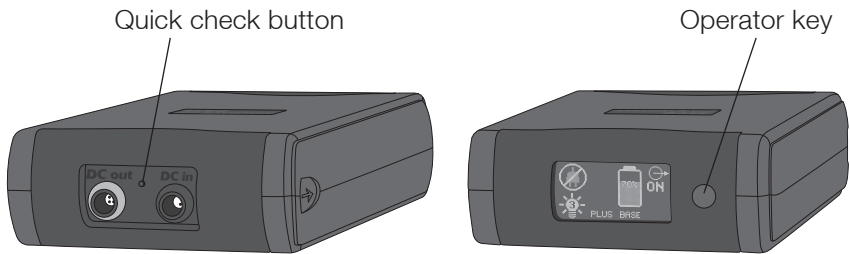
Figure 20: Control elements

SETTING THE DISPLAY BRIGHTNESS

The brightness of the display can be set in 3 levels:

Level 1	Level 2	Level 3
		

Table 5: Display brightness

The brightness level can be changed as follows:

1. Press the operator key for approx. 1 second in order to switch to the Standard screen.
2. Keep the operator key depressed.
3. Once the desired brightness level has been set, release the operator key.

DISPLAYING THE DEVICE IDENTIFICATION AND CHARGE STATUS

In order to check the charge status when a device or power supply unit is not connected, press the quick check button on the rear of the AKKUPACK uni BASE. The serial number and software version are shown for approx. 3 seconds. Subsequently you will see the Standard screen for approx. 25 seconds. Finally AKKUPACK uni BASE then turns itself off automatically.

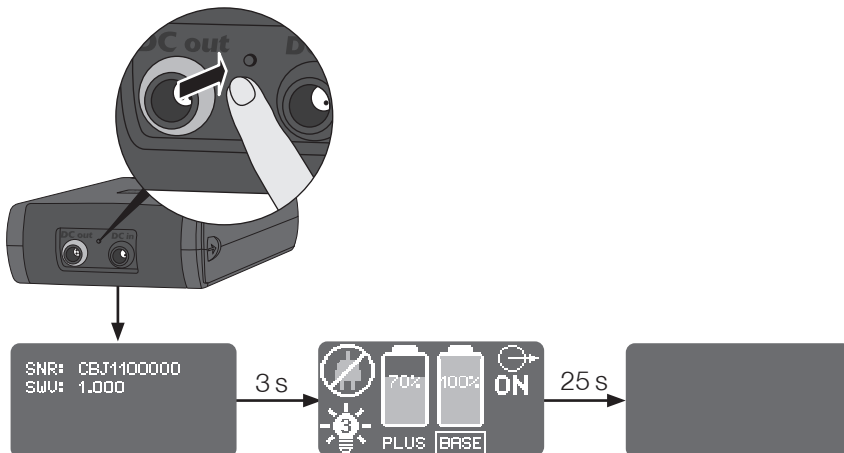
Figure 21: Using the quick check button

BATTERY LIFE

Battery life	BASE	BASE and PLUS
with HOFFRICHTER ventilator		
with following parameters: <ul style="list-style-type: none"> • PCV mode • IPAP = 20 hPa • EPAP / PEEP = 5 hPa • Frequency = 12 bpm • Ramp = 3 • I:E = 1:1.5 • Resistance = 5 • Compliance = 50 ml/cmH₂O 	TRENDvent and CARAT: at least 10 h CARAT pro: at least 8 h	TRENDvent and CARAT: at least 20 h CARAT pro: at least 16 h
with HOFFRICHTER respiratory therapy device BILEVEL ST30		
with following parameters: <ul style="list-style-type: none"> • I-Pres. = 12 hPa • E-Pres. = 6 hPa • Frequency = 14 bpm 	at least 15 h	at least 30 h
with HOFFRICHTER respiratory therapy device CPAP		
with following parameters: <ul style="list-style-type: none"> • Press. = 10 hPa • 4 mm nozzle 	at least 15 h	at least 30 h
The values given are guidelines. The battery life is influenced by other factors including tidal volume, the tubing system used and the number of charging cycles.		

Table 6: Battery lifes

After 300 charging cycles the AKKUPACK uni has around 80 % of its original capacity and around 70 % after 500 cycles.

NOTICE

The battery lifes given in Table 6 can be shorter depending on the number of charging cycles.

When BASE and PLUS are used together, PLUS is always discharged first, followed by BASE. After this, and if present, the internal battery of the connected device is used as the power supply.

Alarm on discharging of the battery	
Alarm sound	Triggering of the alarm
2 x short	at 4 %, 3 %, 2 % and 1 %
2 x long	at 0 %

Table 7: Acoustic alarms on discharging of the battery

NOTICE

Make sure that by using with a ventilator the power supply is still guaranteed.

SEPARATION OF BASE AND PLUS

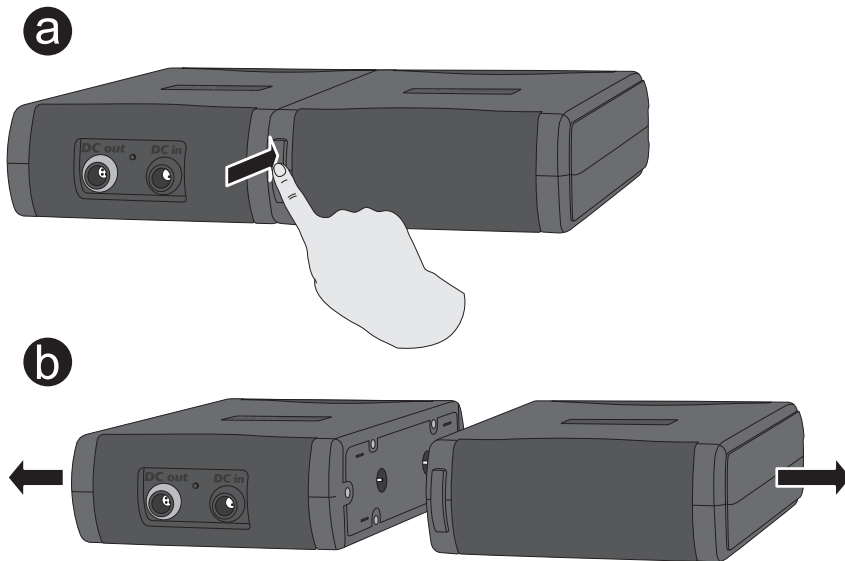
Figure 22: Separation of BASE and PLUS

After separation fit the left-hand side cover as described on page 51.

TRANSPORT AND MOBILE USE

Depending on the scope of delivery, an ever-ready bag for either the AKKUPACK uni BASE or the combination of BASE and PLUS is included. The ever-ready bags have a viewing window for the display and an opening for the DC cable (DC out).

Functional bags are also optionally available. The functional bags are padded and thus offer even better protection against damage. The bags have an additional opening for the mains socket (DC in). This means the AKKUPACK uni can remain in the bag during charging. The bags feature loops on the back for attaching the AKKUPACK uni to a wheelchair, wheeled walker or bag trolley. The functional bags can be purchased from your specialist dealer.

If you wish to use the AKKUPACK uni when traveling by air, please consult your air carrier in advance.

TROUBLESHOOTING

In the event of an error, the Error screen is displayed.

Figure 23: Error screen with error code 206

Figure 24: Error screen with error code 106

With error codes 201 to 399, the AKKUPACK uni BASE can still be operated. All of the errors are displayed one after the other upon pressing the operator key. The Standard screen appears after the last error has been displayed. The error continues to be displayed at the bottom right of the screen until it is eliminated. The Standby screen is not switched to until the error is remedied.

With error codes 100 to 199 it is not possible to switch to the Standard screen.

NOTICE

To allow the AKKUPACK uni to switch off, all errors relating to error codes 100-299 must be viewed beforehand by pressing the operator key.

Error code	Problem/cause	Remedy
100 – 199	Internal device error	Have the AKKUPACK uni checked by customer services.
201	Faulty or non-permissible power supply unit	Check the power supply unit and replace if necessary.
202	Bus voltage too high	Have the AKKUPACK uni checked by customer services
203	Input current is too high / faulty or a non-permissible device is connected.	Check the connected device. If no error can be found, have the AKKUPACK uni checked by customer services.
204	Output current is too high / faulty or a non-permissible device is connected.	Check the connected device. If no error can be found have the AKKUPACK uni checked by customer Services.
206	Short circuit between DC in and DC out	Remedy the short circuit!
266	Update error	Have the AKKUPACK uni checked by customer services.
301	Input voltage too low / faulty or a non-permissible power supply unit	Check the power supply unit and replace if necessary.
303, 353	Charging error	Discharge the AKKUPACK uni until it switches off. If the problem remains, have the AKKUPACK uni checked by customer services.

Table 8: Troubleshooting

CLEANING AND DISINFECTION

- The power plug must be unplugged before cleaning the AKKUPACK uni.
- Aggressive scouring and cleaning agents (e.g. acetone) must not be used for cleaning.
- The AKKUPACK uni must not be immersed in or covered with water or solvents.
- It must be ensured that no water or cleaning agents are allowed to enter the AKKUPACK uni. This could result in malfunctioning, damage and electric shocks.

A cloth moistened with soapy water can be used to clean the exterior of the AKKUPACK uni. Then, wipe with a cloth moistened with clear water, in order to remove any cleaning agent residue.

If used for clinical applications, the surface of the device should be disinfected regularly as well as in the case of suspected contamination. Mikrozyd® liquid is recommended for disinfection of the exterior. The AKKUPACK uni must be completely dry before commissioning.

MAINTENANCE

If there is a noticeable decrease in the battery life in spite of proper charging, it is recommended that the internal batteries in the AKKUPACK uni are replaced. The batteries may only be replaced by the manufacturer or an authorized service company. Ask your specialist dealer about this.

In the case of extended storage periods or non-use, the AKKUPACK uni should be charged every 6 months.

DISPOSAL

Proper disposal saves natural resources and prevents harmful substances being released into the environment.

DEVICE

The AKKUPACK uni must not be disposed of with the household waste. Please contact the relevant customer services department to find out how to dispose of the device properly.

PACKAGING

The packaging is taken back by the distributor but it can alternatively be disposed of with the normal household waste.

ARTICLE NUMBERS

SCOPE OF DELIVERY

Scope of delivery	Article number
AKKUPACK uni BASE “Ventilation”	0001 1100
<ul style="list-style-type: none">• BASE base unit• Side cover, 2x• BASE ever-ready bag• “Ventilation” cable set<ul style="list-style-type: none">– “Ventilation” adapter cable set– Ventilation DC cable• User's manual	4210 1152 0001 1051 0001 1047 0001 4121 0001 1058 5000 0495

Scope of delivery	Article number
AKKUPACK uni BASE “Respiratory therapy”	0001 1101
<ul style="list-style-type: none">• BASE base unit• Side cover, 2x• BASE ever-ready bag• Respiratory therapy DC adapter• Respiratory therapy DC cable• User's manual	4210 1152 0001 1051 0001 1056 0001 1057 5000 0495

Scope of delivery	Article number
AKKUPACK uni PLUS	0001 1099
<ul style="list-style-type: none">• PLUS extension unit• Side cover, 1x• BASE/PLUS ever-ready bag• Instruction sheet	4210 1152 0001 1052 5000 0502

OPTIONAL ACCESSORIES

Accessory part	Article number
BASE functional bag	0001 1049
BASE/PLUS functional bag	0001 1050
Respiratory therapy power supply unit	0000 2020
Respiratory therapy power cable	3110 0015

TECHNICAL DATA

Power supply	
CARAT-/TRENDvent power supply unit	100 ... 240 VAC; 50 ... 60 Hz; 1.4 ... 0.6 A (Input)
	24 V ± 0,5 VDC / 5 A (Output)
	Classification acc. to IEC 60601-1: Protection class II
Respiratory therapy power supply unit	115 ... 230 VAC; 50 ... 60 Hz; 0.79 ... 0.37 A (Input)
	24 V ± 0.5 VDC / 2.1 A (Output)
	Classification acc. to IEC 60601-1: Protection class II
Battery rated output current	4.2 A at 24 V ± 0.5 VDC

Specifications and performance			
	BASE	PLUS	BASE + PLUS
Dimensions in mm (W x D x H)	163 x 205 x 61	166 x 205 x 61	329 x 205 x 61
Weight	1.6 kg	1.5 kg	3.1 kg
Battery technology	Lithium ion		
Battery capacity	2 x 65 Wh	2 x 65 Wh	4 x 65 Wh
Rated voltage	14.8 V		
Switch on threshold values	> 40 mA, ± 10 mA load current		
Switch off threshold values	< 40 mA, ± 10 mA load current		
Protection	Exhaustive discharge, overcharge, excess temperature, overload protection (in- and output), short circuit		
Degree of protection against			
Foreign bodies	IP20 (AKKUPACK uni)		
Water	IP02 (only bag)		
Foreign bodies and water	IP22 (AKKUPACK uni in ever-ready/functional bag)		
Classification acc. to IEC 60601-1	Protection class II		
UN classification	UN 3481 (Lithium ion cells and batteries installed in or packed with equipment)		

Operating, transport and storage conditions	
Relative humidity	15 % to 93 %, non-condensing
Air pressure	700 hPa to 1060 hPa
Charging operating temperature	+ 5 °C to + 40 °C
Discharging operating temperature	- 5 °C to + 40 °C
Transport and storage temperature	- 20 °C to + 50 °C
Storage conditions	Store in a dry, vibration-free place; store device and accessories in their original packaging.

The manufacturer reserves the right to make technical changes without notice.

MANUFACTURER'S DECLARATION ON ELECTROMAGNETIC COMPATIBILITY

The AKKUPACK uni must be operated and used in accordance with this user's manual. Wireless home network devices, cell phones, wireless telephones and their base stations, walkie-talkies etc. can influence the AKKUPACK uni. For this reason, the AKKUPACK uni is to be kept away from such devices.

Example: A typical cell phone with a maximum output of 2 W requires a distance of 3 m for electromagnetic immunity of 3 V/m.

DISCLAIMER

HOFFRICHTER GmbH is not liable for consequences in terms of safety, reliability and performance of the product where:

- interventions, modifications, extensions, calibration, repairs and maintenance are carried out by persons not authorized by us,
- other manufacturers' accessories and spare parts are used that have not been approved by us for use on the product,
- the product is used other than as described in the user's manual or
- the cleaning instructions described in the user's manual have not been complied with.

Statutory guarantee rights remain unaffected by this.