IMPORTANT INFORMATION

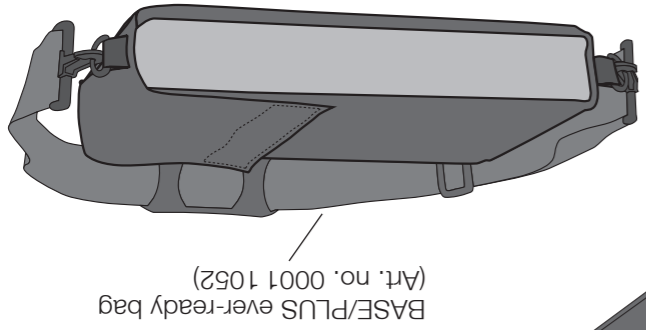
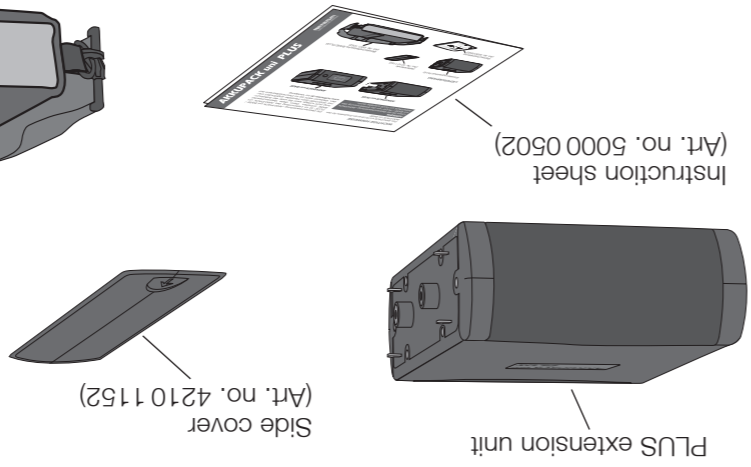
The AKKUPACK uni PLUS is an additional battery for the AKKUPACK uni BASE.

HINWEIS


Read the "AKKUPACK uni BASE / PLUS" user's manual carefully before using the AKKUPACK uni PLUS for the first time.

The "AKKUPACK uni BASE / PLUS" user's manual is included in the scope of delivery of the AKKUPACK uni BASE. If this user's manual is missing, it can be requested from the manufacturer by quoting the article number 5000 0495. It is also available in the download area at www.hoffrichter.de.

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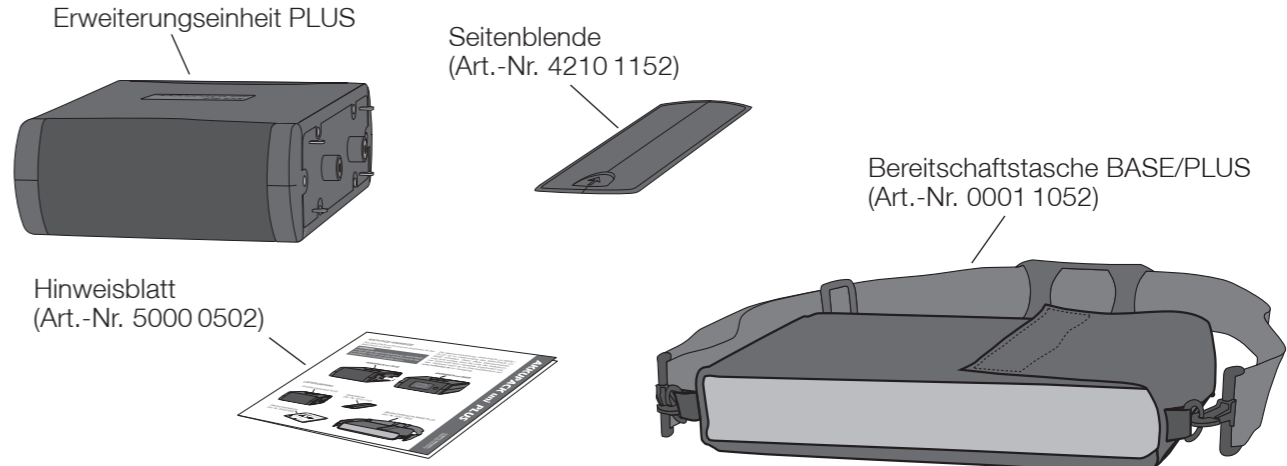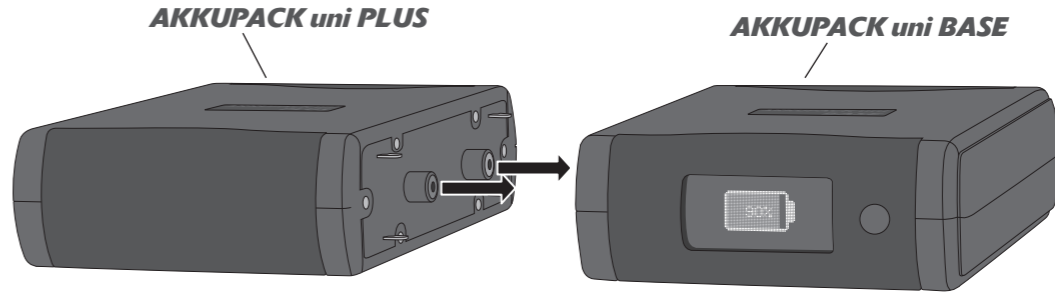
SAFETY INFORMATION

- The AKKUPACK uni PLUS may only used together with AKKUPACK uni BASE.
- The AKKUPACK uni housing does not offer protection against the ingress of water and must never be exposed to rain, moisture or humidity. This could result in malfunction and damage. The AKKUPACK uni must therefore always be in the ever-ready/functional bag, while being transported and operated outdoors.
- Do not place containers with liquid on the device.
- Do not short-circuit the AKKUPACK uni. This could result in damage.
- Only use accessories that have been approved by HOFFRICHTER for use with the AKKUPACK uni.

SYMBOLS ON THE RATING PLATE

Follow the user's manual.

Manufacturer

Serial number

CE conformity declaration

Protection class II

The device is protected against solid foreign bodies with a diameter of at least 12.5 mm.

IP20

Keep dry!

Do not dispose of the device in the household waste. Please contact the relevant customer services department to find out how to dispose of the device properly.

SYMBOLS ON THE BAG

The ever-ready/functional bag is protected against dripping water when tilted up to 15°.

IP02

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HOFFRICHTER GmbH · Mettenheimer Straße 12/14 · 19061 Schwerin · Germany